

Patient Transfer Machine WY-203



To ensure the correct use of the product, please read this instruction manual carefully before use.

- ◆ 4.Follow the manual instructions during disassembly to avoid damage.

●9. Packaging, Transport, and Storage

- ◆ 1.Packaging: Carton packaging.
- ◆ 2.Transport: Handle with care; avoid inversion and exposure to rain.
- ◆ 3.Storage: Store in a well-ventilated, dry area (humidity ≤80%) free of corrosive gases.

●10. Service Life: 5 years

●1. Product Name: Patient Transfer Machine

● 2. Model: WY-203

●3.Structure and Components:


- ◆ Frame, casters, base legs, lifting rod, controller assembly, armrests.

●4. Intended Use:

- ◆ For patient transfer and mobility.

Model	Length	Width	Height of casters	Seat Height	Overall Height
WY-203	80cm	54cm	16cm	50-70cm	97-117cm

●5.Assembly Instructions

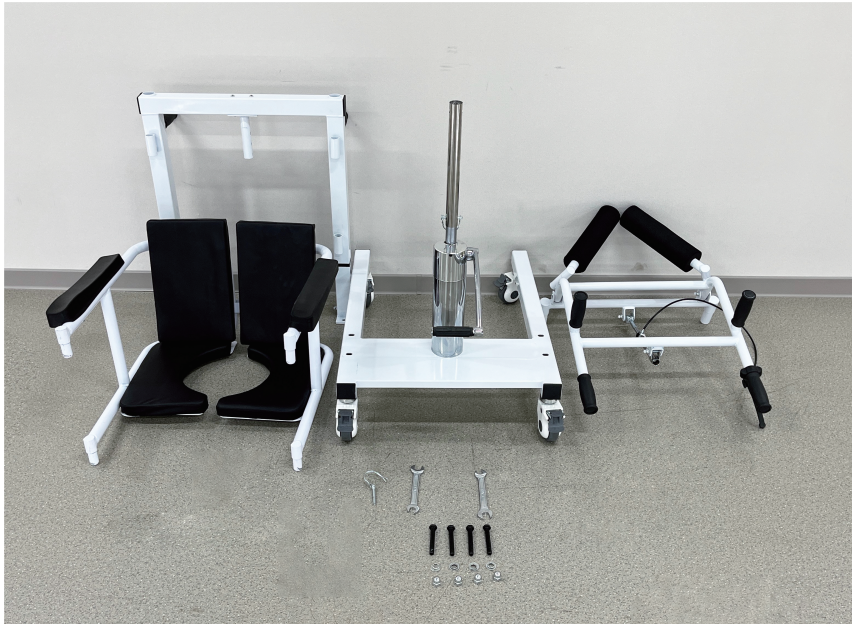

Bill Of Material



1. Place the U-shaped bracket above the chassis. Align the white rod in the middle of the U-shaped bracket with the vertical pipe above the hydraulic pump and insert it into the vertical pipe. Insert the screw from the bottom and tighten it with a wrench.



2. Adjust the height of the U-shaped bracket so that the two holes are at the same level. Then insert the pin and lock it.



3. Insert the left and right seat plate rotating shafts into the corresponding shaft holders respectively.



4. Rise up and insert the bracket into the corresponding installation hole.

● 6. Operating Procedures

Seated Transfer:

- ◆ 1. Have the user sit upright. Open the seat and position it in front of the user. Lift the user's feet onto the footrest and lock the brake before seating. The parking brake lock is locked before the user sits on the transfer machine.
- ◆ 2. Guide the user to lean slightly left/right while rotating the corresponding seat half to allow entry. Close the backrest and lock the automatic latches and safety latches once fully seated.
- ◆ 3. After confirming both backrest latches are locked, release the brake. Move the user using the front handle.
- ◆ 4. Upon reaching the target location, open the dual backrest latches, rotate the seat open, and assist the user in exiting.

Height Adjustment:

- ◆ 1. Loosen the locking lever and adjust the seat height by moving the control handle up/down.
- ◆ 2. Tighten the locking lever once the desired height is achieved.

● 7. Safety Precautions

- ◆ 1. Before use, ensure all components are complete and secure for safety.
- ◆ 2. Do NOT stand on the footrest during entry/exit; always step on solid ground.
- ◆ 3. Adjust the machine to a suitable height before use.
- ◆ 4. Ensure both seat halves are at the same height before use.
- ◆ 5. Do NOT adjust the height while the user is seated.
- ◆ 6. Lock all wheels during entry/exit.
- ◆ 7. Lock all four wheels when stationary on slopes.

● 8. Maintenance

- ◆ 1. Regularly inspect and maintain the machine to prevent user injuries.
- ◆ 2. Clean stains with neutral detergent.
- ◆ 3. Periodically check for loose nuts, damaged, or broken parts; replace if necessary.